Bionova® MiniPro Auto-reader

Symbols | Símbolos | Símbolos | Simboli

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EN Bionova® MiniPro Auto-reader

Product description

The Bionova® MiniPro Auto-reader was designed for the automatic readout of Terragene Chemdy® Hygiene Monitoring System indicators (Protein Pens) appropriate for the detection and quantification of surface proteins. The Bionova® MiniPro Auto-reader incubates at 60 °C and performs a quantitative analysis over the Chemdy® Hygiene Monitoring System Indicators (Protein Pens), adjusted by a BSA (Bovine Serum Albumin) reference curve. The Bionova® MiniPro allows the incubation and readout of three independent Chemdy® Hygiene Monitoring System Indicators (Protein Pens). The Bionova® MiniPro Auto-reader includes a detection limit of 0.5 µg and a quantification range from 1 to 50 µg of equivalent BSA Protein. The results are indicated with a resolution of 0.1 µg, with an accuracy greater than 90 % in the whole quantification range (C95 %). The Bionova® MiniPro Auto-reader also provides a printed ticket each time an automatic readout is completed for recording the results. This allows for easy result management, for documentation compliance and safekeeping. **Note:** Do not place a new Protein Pen until the temperature is stable and the Temperature Stability Indicator stops blinking.

Indications for Use

The Bionova® MiniPro Auto-reader incubates at 60 °C and reads the Terragene Chemdy® Hygiene Monitoring System Indicators at the times prescribed in the User Manual.

Safety Information

⚠️ To avoid risks and/or damaging the device:
 _Use indoors only.
 _Do not place the auto-reader in a room exposed to direct sunlight or to high luminous intensity lamps.
 _Do not place the auto-reader near devices that emit strong electromagnetic fields.
 _Do not use the auto-reader on sloping surfaces or on surfaces that might be knocked, that may vibrate, be exposed to high temperatures, or high relative humidity.
 _Disconnect the power cord before cleaning.
 _Do not use abrasive, corrosive cleaners or disinfectants.
 _Do not immerse into any liquid. Do not pour liquid into it.
 _Make sure the auto-reader is connected to an appropriate electrical mains outlet socket.
 _Use only the included power supply (AC power adapter), power supply/s AC plug, power supply cords, and USB cable. Check that all the items included are in good condition every day. If any of them is damaged, stop using them. The use of cables, adapters, cords, and/or power supplies different from the ones included may cause fires, electrical shocks, or even physical injuries.
 _Do not plug any devices into the auto-reader's USB port other than a personal computer (PC). The PC must be compliant with IEC 60950-1, IEC 62368-1 or comparable, with safety extra-low voltages on its USB ports. Ask a qualified technician to check that the device is compatible. Attaching any other device to the USB port may damage the auto-reader and may not be safe for its user.
 _Do not attempt to repair the auto-reader yourself. This could lead to major and irreversible damages to the device. If the auto-reader malfunctions, contact your local distributor for further assistance.

⚠️ To reduce the risks associated with contaminated surfaces with protein levels outside the user-defined threshold:
 _Please read, understand, and follow the instructions for Use of each Protein Pen before its incubation.
 _Do not remove the Protein Pen before the auto-reader reports the final readout result. Check the printed ticket.

⚠️ To avoid a potentially hazardous situation:
 _Avoid contact with the hot metal block inside each incubation position.
 _Do not insert your fingers, or any other element, into the incubation positions.
 _Place only compatible indicators inside the incubation positions.

Note: Do not use this product in a manner not specified by Terragene S.A., otherwise the protection provided by the product might be affected.
Note: Only Terragene S.A. authorized personnel may access or service the internal components of the auto-reader. Parts or components inside the auto-reader should not be handled by the user.

Operating conditions Power supply specifications		
Input parameters	Operating conditions	Units
Voltage range	(100-240)	AC Volts
Frequency	50-60	Hertz
*Current	0.5	Amperes
Output parameters	Values	Units
Voltage	12	DC Volts

Output parameters	Values	Units
**Current	2	Amperes
* 1 Ampere in USA and Canada * 3 Amperes in USA and Canada		

Terragene S.A. recommends the use of Uninterruptible Power Supply (UPS) instead of voltage stabilizers since they fulfill two functions: To stabilize and maintain the energy during a power outage.

Environmental operating conditions		
Environmental conditions	Operating conditions	Units
Altitude	3500 (max.)	Meters
Operation temperature	10-30	Celsius
Relative humidity	30-80	%
Installation/Over-voltage	Category II	-
Pollution degree	2	-
Storage temperature	5-40	Celsius
Voltage	12	DC Volts
Device safety compliance		
The auto-reader complies with the following standards and directives:		
Electrical Safety	IEC 61010-1 IEC 61010-2-010 Low Voltage Directive 2014/35/EU	
Electromagnetic Compatibility (EMC)	EN 61326-1 EMC Directive 2014/30/EU RoHS Directive 2011/65/EU WEEE Directive 2012/19/EU	
The Bionova® MiniPro Auto-reader does not represent photobiological risk and does not generate dangerous optical radiation if used within normal operation conditions as per the requirements of IEC 62471 Standard.		

Designed under Quality Management System standards ISO 13485:2016 / EN ISO 13485:2016.

Cleaning and maintenance
Cleaning and decontamination of external surfaces
 Disconnect the power supply cable and the USB cable from the auto-reader. If the device is hot, wait until it has cooled down before handling it. Clean the external surfaces of the device using a microfibr cloth moistened with the solution of mild dish washing detergent and water. Wring out the cloth so it is damp but not dripping before cleaning and wipe the outer surfaces of the auto-reader. Then, moisten a clean microfibr cloth with just water, and repeat the procedure until all traces of detergent are removed from the external surfaces of the device. After cleaning, allow the auto-reader to air dry for at least 1 hour before connecting the power supply cable to the USB cable again. This cleaning procedure can be followed whenever considered appropriate. This cleaning procedure must be followed each time a spill occurs on any of the external surfaces of the auto-reader. If further cleaning is required, or if you have doubts about the cleaning agents you may use, please contact your local distributor.

Note: Do not clean the internal parts of the device.
Note: Do not pour liquid on to the device or immerse it in any liquid. Do not allow any liquid to enter the device while it is being cleaned.

Maintenance
 The Bionova® MiniPro Auto-reader does not require routine maintenance.

Warranty
 The products are guaranteed to be free from material and workmanship defects when properly installed, maintained and used for their intended purpose indicated in the applicable product label and/or the User Guide. The warranty only applies to the original purchaser.

Term
 The warranty period for the Bionova® MiniPro Auto-reader is 1 (one) year from the date of installation and may never be extended beyond 5 (five) years from the product's date of manufacture.

Limitation of liability
 Terragene S.A. shall not be held liable for any loss or damage that result from the unsuitable use of the equipment, negligence or user's full responsibility.

Technical assistance
 Terragene S.A.
 Ruta Nacional Nº 9, Km 280 - CP 2130. Parque Industrial Micropi-Alvear-Santa Fe-Argentina.

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EN Instructions for use

Setting Up your auto-reader

Place the auto-reader on a firm surface, free from vibrations, away from direct sunlight, currents of hot or cold air, chemical and corrosive or flammable substances. Do not place the auto-reader in a way that makes disconnecting the power supply's AC plug from the mains difficult. Leave a space of at least 10 cm between the auto-reader and the closest wall. Do not move the auto-reader periodically or during its use. Connect the auto-reader to a secure and stable electrical mains outlet socket. Do not wet or heat the auto-reader. If liquid is spilled on the device, disconnect it and dry it immediately. Before turning it on, verify that all reading positions are empty.

Using your auto-reader

- 1 | Switch on your auto-reader by connecting the power supply's AC plug to the mains and then connect the plug at the other end of power supply to the rear of the auto-reader. (See image 1)
- 2 | Wait until the Temperature stability indicator stops blinking for the auto-reader to reach a steady working temperature. Press the Start incubation program button of the selected position for 1 second to enter the reading mode. (See image 2)
- 3 | Wait until the Quantitative protein level indicator stops blinking. Place the Protein Pen in the corresponding position. Press the PRO button again for 1 second to start the incubation. (See image 3)
- 4 | Wait until the incubation is completed. The Green light indicator will turn on in case of a negative result. The Red light indicator will turn on in case of a positive result. If a positive result is detected, press the Alarm cancellation button to cancel the alarm. Do not remove the indicator until the ticket with the result of the incubation is printed. (See image 4)

Disposal: Dispose of the Protein Pens in accordance with your country's waste-disposal guidelines.

Connectivity

To connect your MiniPro Auto-reader to a PC via a USB port, connect an end of the USB cable (included) to the rear part of the incubator and the other end to the USB port of your PC. Connecting the auto-reader to a PC allows the use of the Bionova® Traceability. Please refer to the User Manual for more information.

Bionova® MiniPro Auto-reader Incubation Programs

The following incubation program is available for your auto-reader.

Temperature selection	Time selection	Incubation program
60 °C	4 minutes	4 minutes at 60 °C

Follow each indicator's instructions for use before its incubation.

Troubleshooting

Refer to the user manual for a detailed troubleshooting routine. Please visit www.terrogene.com/MINIPRO or scan the QR code to access the auto-reader user manual.

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